



Prepare Device and Cover

1 Turn off the device

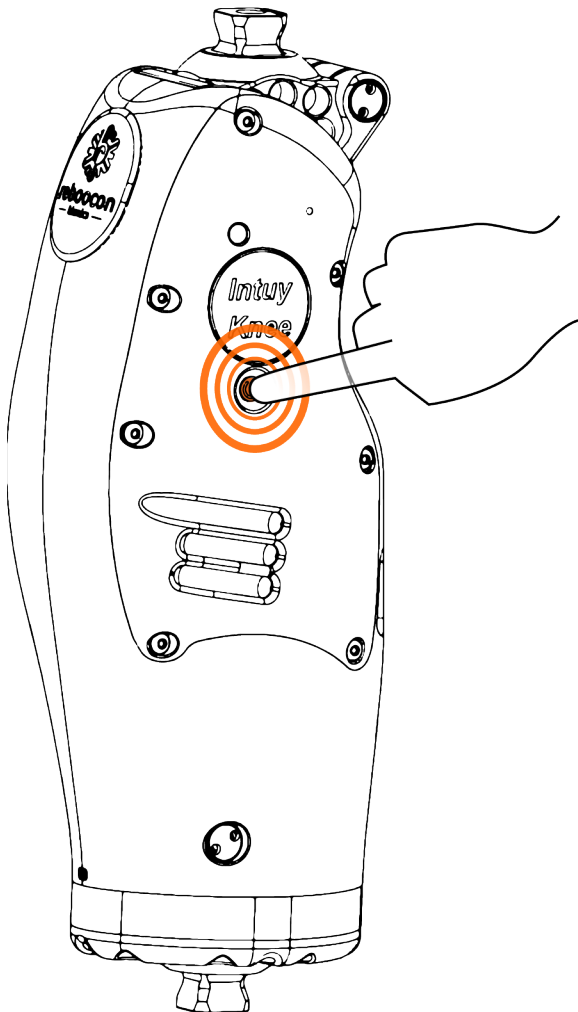


2 Add silicone oil on the device

Caution: Be sure to avoid getting silicon oil on the coil.

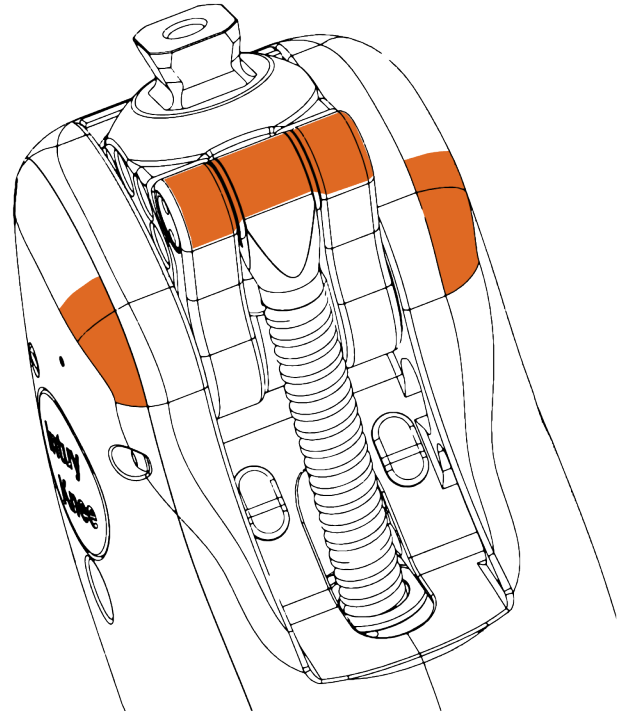


3 Add silicone spray inside the cover

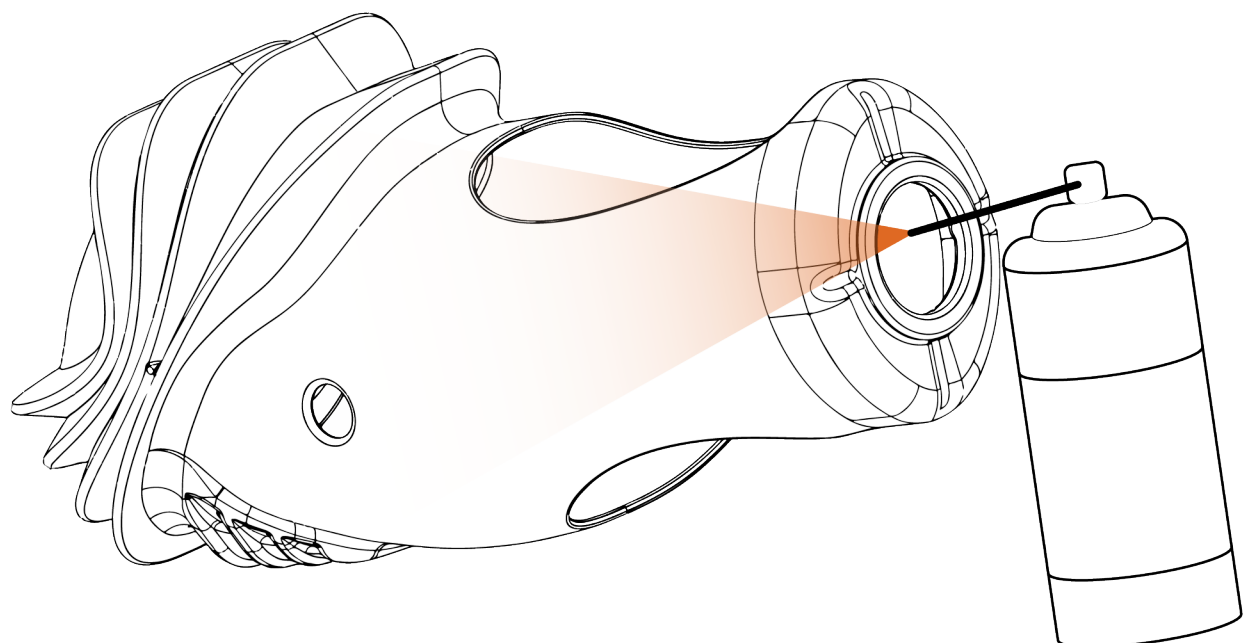


Install Cover on Intuy Knee


4 Insert the knee in the cover

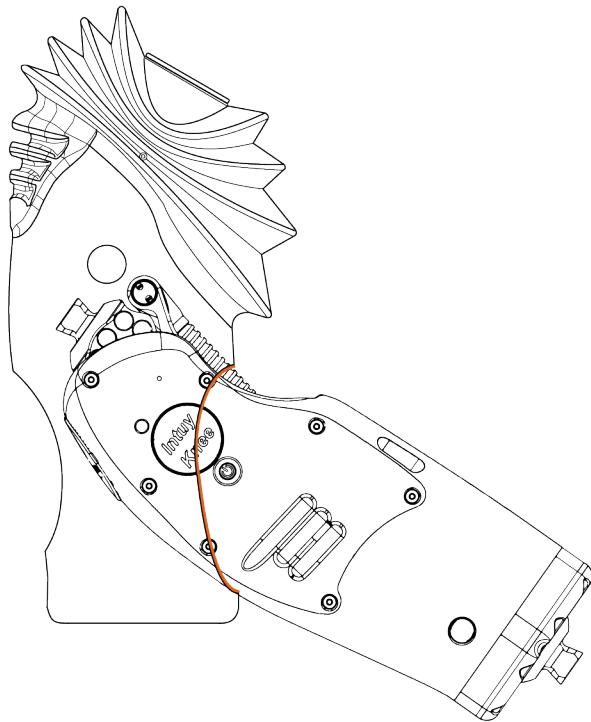


5 Start by aligning the top

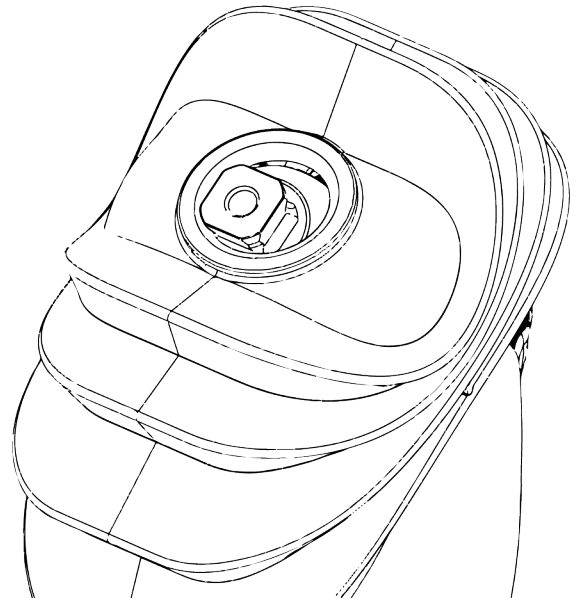

6 Align the bottom

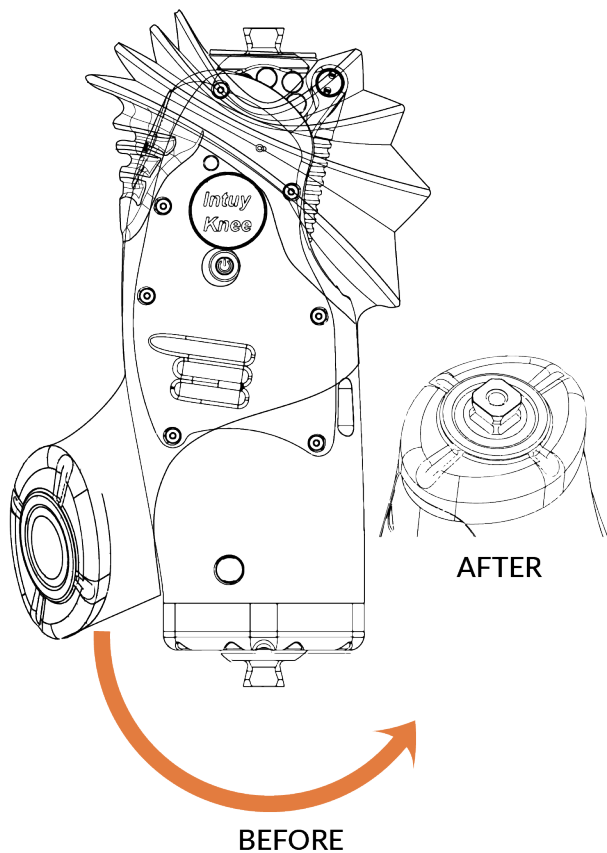


7 Check the alignment

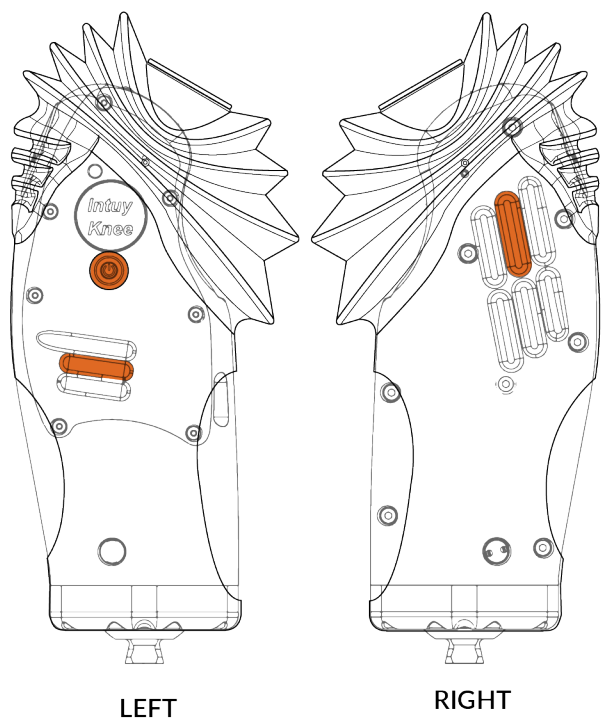


Assemble the Prosthetic Leg

Note: Before the assembly **Lock** the Intuy Knee from the app. During assembly, make sure the cover does not get jammed between the knee and the adapters.


- 8 Add silicone to the cover**
This reduces the chance of jamming the cover.

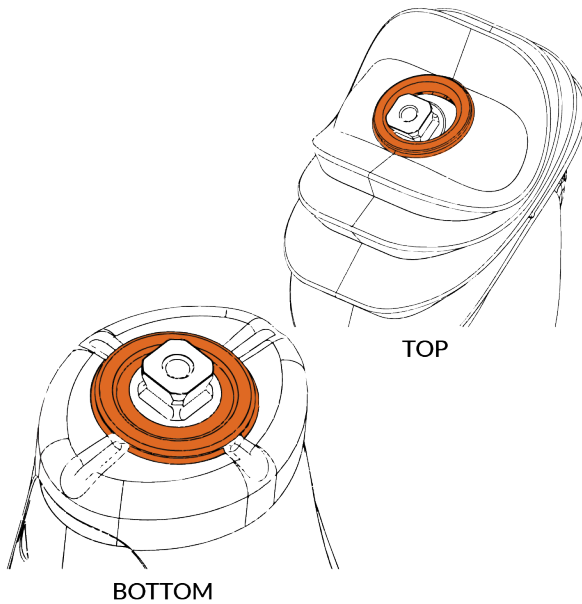

- 9 Connect foot to the tube**
Keep the shoe on. Place the foot in a neutral position.

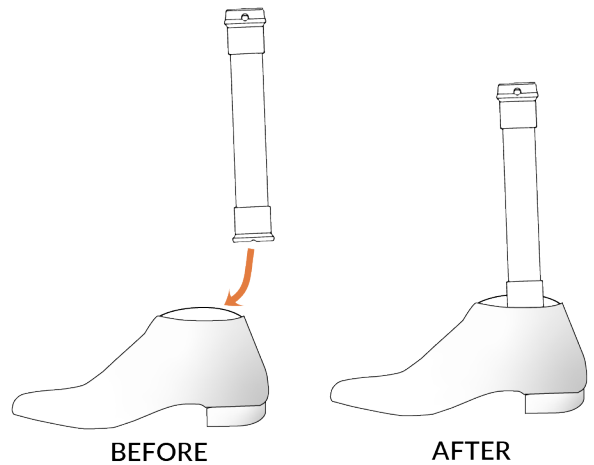

- 10 Connect the knee with foot**
Use a torque wrench to tighten all four screws equally based on the specifications of the pylon adapter.

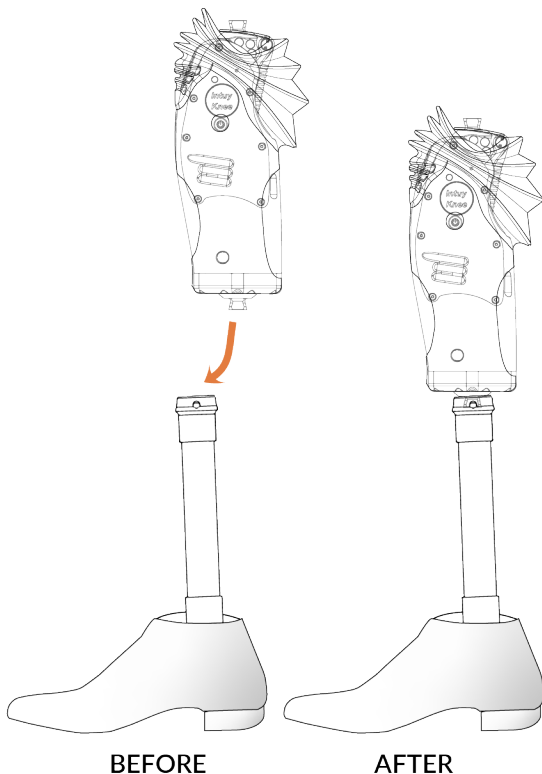

- 11 Connect knee with socket**
Connect the socket adapter to the top of the knee in a neutral position.

