

Maintenance Reminder Muting:

Temporarily mute maintenance reminders for devices with low movement count, loaners, or stock devices. Muting maintenance reminders can be done remotely or with the device on hand.

Keep in mind, that movement-based maintenance reminders will still be active. If the device reaches the set limit of movement for maintenance, reminders will start again.


Note: Only available for Intuy Knee firmware 2.5.13 and Intuy App 4.10.1

Steps to mute maintenance reminder


1 ☰ Tap Menu button.



2 📦 Open Device Management

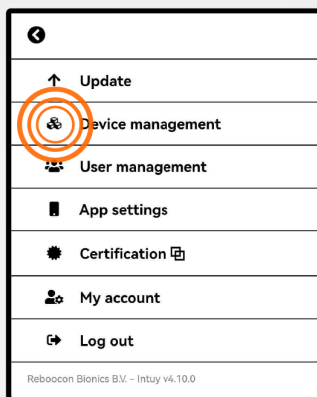

3 🔍 Search for device

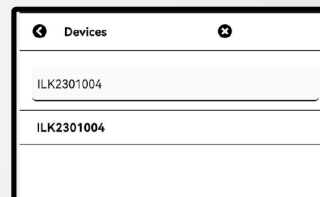

If **change is done remotely**, please continue with these steps:

- 5 📶 Connect with Intuy Knee
- 6 📶 Have a internet connection
- 7 🏠 Muting status is now indicated on the Dashboard notification.

📌 12:28:52 - Time-based maintenance reminder is muted. Step count-based reminder is still active.

4 🗑️ Toggle off reminder

